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EN					
<b>INSTRUCTIONS FOR USE / INFORMATION ON THE PRODUCT</b>					
<b>DESCRIPTION</b>					
<b>Medgel</b>					
Medgel has been specifically formulated for the treatment of keloids, scars, hypertrophic scars and associated with erythema (redness).					
The Medgel is made from medical grade silicone gel, a biocompatible material that provides a unique combination of hydration, comfort and skin care. It has a soft grip, is flexible and reusable. Can be used under clothing and compression garments.					
<b>Advanced technology</b>					
Medgel aids in the process of hydration of the healed tissue, providing an improvement in the appearance and smoothness of scars, keloids and hypertrophic scars.					
Due to its flexible mechanical characteristic, it is able to be adapted to the contour of the skin, sticking gently to the entire region of the scar. Medgel can be re-used after washing.					
Silicone (Medgel's constituent material) when used on recent scars reduces the incidence of hypertrophic scars and reduces the thickness of post-traumatic hypertrophic scars.					
Its continuous use provides an improvement in the pigmentation, vascularization and flexibility of the scar.					
<b>Use Indication</b>					
Medgel products SILIMED are intended for use in the treatment of scars, hypertrophic scars and keloids. Help in relieving of the discomfort and are easily applicable and removable. Many old scars may also benefit from Medgel's application.					
Effective in the aid of the treatment of scars resulting from incisions made in surgeries, scars from tattoo removal, burns, acne, cuts, animal bites, among others.					
It is up to the physician to select the appropriate type to meet the clinical and aesthetic requirements of each case:					
Drawing	Reference	Shape	Units per package	Size	
	1240-01312	Inverted "T"	2	34 x 9,5 cm	
	1240-010H 1240-010I1 1240-016I1	Plate	6 1 1	10 x10 cm 10 x 10 cm 10 x 5 cm	
	1240-011H 1240-011H1 1240-011I2		Strip	12 1 2	30 x 2 cm 30 x 2 cm 30 x 2 cm
	1240-012H 1240-012I2			Disk	12 2
	1230-015	Coat			1

**Composition –** Medgel is compos of silicone of grade physician biocompatible.  
**Medgel** (Plate, Strip, Disc) ----- Silicon gel and polydimethylsiloxane elastomer, catalyzed by platinum complex.

**Medgel Coat:** ----- Silicone Gel with copolymer of Polydimethylsiloxane.

**Obs:** The silicone may exhibit a color variation, ranging from opaque gray to translucent yellow. he Medgel (Plate, Strip, Disk) may exhibit a variation of adhesiveness.

**Presentation**  
Provided non-sterile. The Medgel (Plate, Strip, Disc) is protected by plastic and packed in plastic packaging, containing one, two or twelve units according to the references. Medgel Coat is supplied in individual packaging - 15g tubes and external packing of paperboard

**Contraindications**  
The use of Medgel Coat is contraindicated for open wounds, unsound tissue and on infected regions.

**Precautions**  
Avoid Medgel coming into contact with the eyes and mucosas. Take care when using on recently injured tissues. Keep this product out of the reach of children. Medgel shall only be applied on clean, dry skin, and it shall not be used together with lotions, ointments, hydrating or any other skin-treatment products, since they can interfere in the performance of the product and/or cause skin rash. Some studies have reported that patients using silicone gel to treat scars present few adverse effects in the region covered by the product, such as skin irritation (possibly presenting itching, contact dermatitis and rash) and skin maceration. If any of these effects occurs, it is preferable to use the Medgel for a few hours, alternating with periods of rest. In case the rash persists, the use shall be discontinued.

**Instructions for Handling and Use**  
Is indicated to keep the Medgel on the scar for at least 12 hours a day.  
To prevent the occurrence of hypertrophic scars, Medgel must be applied immediately after the last "scab" has fallen off the surgical scar.

For Plate, Strip, Disk :  
1 - Gently wipe affected area and dry.  
2 - Remove Medgel from its envelope.  
3 - Cut a piece big enough to cover the scar completely (in case of the Plaques and Strips).  
4 - Remove the glued protection from the side without the reinforcement, which is adhered to Medgel.  
5 - Apply this side of the Medgel to the scar, without stretching it.  
6 - If necessary, secure Medgel with a hypo-allergic bandage or elastic gauze. Do not squeeze the silicone plaquenear the skin so as not to cause any irritation of the scar and adjacent area. Medgel can also be used under a compression garment if it is convenient.  
7 - Wash the Medgel every day with neutral soap; rinse with lots of lukewarm water; dry in the air and apply again to the scar. Use no detergent, in order to avoid the possibility of a skin reaction.  
8 - Prior to re-applying the Medgel, make sure that the scar and the surrounding area are washed and dry, as well as the Medgel; if necessary dry it with a soft, lint-free towel.  
9 - Following the above instructions correctly, it is advisable to change the Medgel (Plate, Strip, Disk and Inverted "T") every 20 days to avoid deterioration. The change can be more frequent if the scar is located at a joint or where friction is repeated.

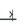
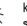
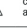
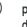
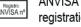
For Medgel Coat:  
1 - Wash and dry the affected region gently.  
2 - Apply a small amount of Medgel Coat, smearing over the desired region Once exposed to the air, Medgel Coat forms an invisible membrane that is permeable to air but waterproof.  
3 - Apply twice a day. An excessive amount of the product may stain clothes.

**Important Recommendations / Warnings**  
•The product and its packaging shall be intact, otherwise the product shall not be used. It can only be used if found whole in its original manufacturing state, that is, with no alterations in its original characteristics.

•The surface of the Medgel (Plate, Strips, Disks and Inverted "T") shall not be contaminated by talc, dust, lint or oils.  
•Keep the Medgel in a clean, dry and well-aired place. Preferably, keep the piece of tape in use in its original packaging during the treatment resting period.

•It is not supplied sterile, it is up to the patient to clean the product under the conditions of its use. The Medgel (Plates, Strips, Disks) should be washed only with neutral soap under running water. Do not use brushes or other abrasive instruments to wash it.

**Durability –**  
It is advisable to change the Medgel (Plate, Strip and Disc) every 20 days, thus avoiding its deterioration. The change can be more frequent if the scar is located at a joint or where friction is repeated, or if deemed necessary.

 dimensions of product	 manufacturer	 keep dry	 keep away from sunlight	-
 date of manufacture	 use by	 lot number	 caution, consult accompanying documents	-
 reference number	 quantity of product	 do not use if package is damaged	 non sterile	 ANVISA registration

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