Camphorate Paramonochlorophenol

ENGLISH

Content:

Bottle with 20 ml.

Indication:

The CAMPHORATE PARAMONOCHLOROPHENOL is an antiseptic product used as an intracanal medication (temporization).

Product Description:

The CAMPHORATE PARAMONOCHLOROPHENOL is a colorless or pale yellow liquid with a camphor aroma. It has antiseptic properties of phenol and chlorine, and has a broad antibacterial spectrum, being also effective against fungi. Camphor is a vehicle and also decreases the irritant action of the phenolic derivative, resulting in a low-power aggression substance on the living tissue when used the shown concentration.

Composition:

Paramonochlorophenol, camphor, ethanol and purified water.

How to Use:

1. Instrument and dry the root canal.
2. Soak a cotton ball in the product and dry it with a small cotton roll.
3. Place the dry cotton ball in the pulp cavity and close tightly with appropriate cement.
4. The estimated time of use is 24 to 48 hours. Then, remove the bandage and carry out root canal filling.

Precautions:

- Product for dental surgeons use only.
- Keep out of reach of children.
- Do not use the product with expired expiration date.
- Non-sterile.
- This product is manufactured for dental use and should be handled according to instructions.

Transport and Storage:

Store tightly closed in a clean, fresh and light-protected place.

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